



Lipoderma[®]

Procedure Checklist

Prior to Scheduled Procedure Date

Please ensure the following materials are available prior to the procedure date:

- Product Preparation Guide
- 18-gauge needle for withdrawal of Lipoderma from the vial / jar
- 21-gauge or larger needle or cannula for procedure

Supplies can be ordered from vendors such as:

- [Face Med Store](#)
- [MedLine](#)

Day of Procedure

- Remove product from storage
 - Minimum **30 minutes** prior to the procedure for the 1.5 cc size
 - Minimum **60 minutes** prior to the procedure for the 10 cc size
- Maintain product labels in the patient's file
- Complete the Allograft Utilization Record
- Follow the instructions outlined in the Product Preparation Guide
- Reach out to your Lipoderma representative with any questions prior to beginning the procedure
- If planning to post to social media, please refer to the Social Media Guidance

Questions?

Please reach out to:

Lisa Rodemann, Director of Customer Experience

- Email: lrodemann@britecyte.com
- Phone: 443-677-6953