



1. Gather Supplies

Prepare for Lipoderma implantation with sterile gloves, one sterile syringe, and one sterile 18G or larger beveled needle for withdrawal.



2. Inspect Packaging

Remove Lipoderma package from frozen or refrigerated storage. Inspect the package to ensure its integrity. **DO NOT USE** the product if the package integrity is compromised.

Note: Jar is aseptically packaged in two presterilized pouches.



3. Thaw Vial

Remove outer pouch and place the inner pouch with the vial onto a clean surface. Allow the product to thaw completely before use (approx. 60 minutes), if needed.

DO NOT RE-FREEZE the product.



4. Retrieve Jar

Retrieve the jar from the innermost pouch. Use aseptic technique to transfer the jar onto a clean surface or a sterile field.




5. Mix Contents

Shake the jar for 15-20 seconds and "wrist flick" the jar in the downward direction to ensure all product is on the bottom of the jar.



6. Withdraw Product

Immediately withdraw the product using a syringe with an 18G or larger beveled needle at a slow, steady pace. **Note:** If the needle becomes clogged, expel clogged tissue. Repeat steps 5 & 6.

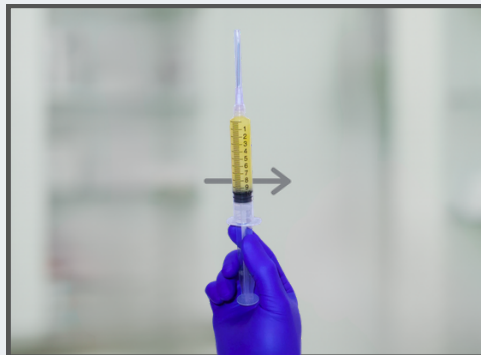

7. Replace Needle

If desired, once the product is drawn into the syringe, replace the 18G beveled needle with a new needle or cannula. Remove air bubbles from the syringe prior to implantation.

DO NOT USE anything smaller than a 21G needle or cannula.

Important Note: Keep Lipoderma in the sealed package until ready for use. Once the package is opened, use within 4 hours. If not used within 4 hours, product must be discarded.

